KAYA  
SAFETY

KAYASAFETY.COM

**RL-10**  
KONİK SEDYE  
CONIC STRETCHER

**KULLANIM KILAVUZU**  
USER GUIDE

**A**

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**(EN)**

Thank you for purchasing Kaya Safety Conic Stretcher. You will find it a valuable tool that will enable rescues to be performed safer, faster, and easier. Our Conic Stretcher is an entirely different concept in stretcher design. Therefore, potential rescuers should practice and familiarize them selves prior to using it on an actual rescue. This stretcher provides excellent patient support and protection however if a spinal injury is suspected, secure the patient to an approved spinal immobilization device prior to placing on the stretcher. The stretcher must be used with its original accessories provided in the package.

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(EN) Continue dragging until his upper chest is even with the upper cross strap.

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(EN) Secure all buckles and adjust all straps in preparation for transport.

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(EN) Fasten The Straps and Adjust Buckles

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(EN) Prior to loading the stretcher release all buckles and pull straps to one side.

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(EN) Lift sides of stretcher and fasten the four cross straps to buckles directly opposite to the straps and pull them tight.

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(EN) Fasten foot end buckles and adjust straps.

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(EN) Do not criss-cross straps. It will cause the stretcher to bend in the middle.

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(EN) Continue threading the rope on each side of the stretcher as shown in picture above.

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(EN) Thread the rope through the bottom grommets from the inside out as shown above.

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(EN) Tie the rope in a square knot at the lower end of the stretcher.

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(EN) Pass the ends of the rope through the lower carry handles from the outside-in and tie another square knot.

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(EN) Tie an overhand knot on each side of the square knot for safety.

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(EN) Hold the pre-tied figure 8 knot in the center of the rope and pass the ends of the rope through the angled lift sling slots from the inside-out.

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(EN) Pass the ends of the rope under the stretcher and through the angled lift sling slots on the opposite side from the outside in.

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(EN) Tie figure 8 knots in both ends of the rope leaving about 2.5 feet extending up from the foot end slots and shorter at the head end.

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(EN) The rope at the head end must be a bit shorter than the foot end to provide a slightly "head up" attitude for patient's safety and comfort.

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(EN) Lay stretcher out flat with all cross straps connected. Place retain-er strap with inside of buckle facing up under foot end of stretcher.

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(EN) Starting at the head end, roll the stretcher up as tight as possible. Continue to roll the stretcher using knee to keep the rolled stretcher small.

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(EN) Fasten the preplaced retainer strap to buckle and place stretcher in backpack.

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EN 1865

### **D- Accessories**

### **E- To Unroll Conic Stretcher and Lay Flat**

- E-1-** Unpack the Stretcher and place it on the ground.
- E-2-** Release the buckle on the stretcher and lay on the floor.
- E-3-** Bend the stretcher over your forearm and back roll. Repeat with opposite end of stretcher.
- E-4-** Stretcher will now lay flat.

### **F- To Place A Patient On The Stretcher Pull Method (Feet First)**

- F-1-** Prior to dragging a patient onto stretcher, Curl the head end downward to form a ramp and to prevent snagging patient's clothing.
- F-2-** Tuck the drag webbing under the stretcher, hold patient's legs and drag patient while keeping legs low.
- F-3-** Continue dragging until his upper chest is even with the upper cross strap.
- F-4-** Secure all buckles and adjust all straps in preparation for transport.

### **G- To Place A Patient On The Stretcher Pull Method (Head First)**

- G-1-** Curl foot end of the Stretcher downward to form a ramp and pull foot end straps to the side.
- G-2-** Using your forearms to prevent lateral movement of patient's head, grasp the underarms and drag inline with the spine onto the stretcher.
- G-3-** Continue dragging until his upper chest is even with the upper cross strap.
- G-4-** Secure all buckles and adjust all straps in preparation for transport.

### **H- Fasten The Straps and Adjust Buckles**

- H-1-** Prior to loading the stretcher release all buckles and pull straps to one side.
- H-2-** Lift sides of stretcher and fasten the four cross straps to buckles directly opposite to the straps and pull them tight.
- H-3-** Fasten foot end buckles and adjust straps.
- H-4-** Do not criss-cross straps. It will cause the stretcher to bend in the middle.

### **I- Horizontal Lift / Decent**

- I-1-** Insert one end of head strap through angled slot at head end of stretcher.
- I-2-** Bring strap under stretcher and through angled slot on opposite side.
- I-3-** Equalize straps. Repeat procedure with other strap at foot end of the stretcher.
- I-4-** Equalize all four straps and secure ends with steel locking carabiner. Your stretcher is now ready for horizontal lifting.

### **J- Vertical Lift / Decent**

- J-1-** Prior to hoisting. The head end must be curled over the patient's head and the drag handle webbing tied to the second cross strap to protect patient's head from falling objects.
- J-2-** Insert vertical lift sling rope through the grommets above patient's head.
- J-3-** For extremely small openings, leave the head end extended. Start threading the rope through the grommets by the patient's shoulders.
- J-4-** Continue threading the rope on each side of the stretcher as shown in picture above.
- J-5-** Thread the rope through the bottom grommets from the insideout as shown above.
- J-6-** Tie the rope in a square knot at the lower end of the stretcher.
- J-7-** Pass the ends of the rope through the lower carry handles from the outside-in and tie another square knot.
- J-8-** Tie an overhand knot on eachside of the square knot for safety.

### **K- Horizontal Lift With Vertical Lift Sling**

- K-1-** Hold the pre-tied figure 8 knot in the center of the rope and pass the ends of the rope through the angled lift sling slots from the inside-out.
- K-2-** Pass the ends of the rope under the stretcher and through the angled lift sling slots on the opposite side from the outsidein.
- K-3-** Tie figure 8 knots in both ends of the rope leaving about 2.5 feet extending up from the foot end slots and shorter at the head end.
- K-4-** The rope at the head end must be a bit shorter than the foot end to provide a slightly "head up" attitude for patient's safety and comfort.

### **L- Rolling Up The Stretcher**

- L-1-** Lay stretcher out flat with all cross straps connected. Place retainer

strap with inside of buckle facing up under foot end of stretcher.

- L-2-** Starting at the head end, roll the stretcher up as tight as possible. Continue to roll the stretcher using knee to keep the rolled stretcher small.
- L-3-** Fasten the replaced retainer strap to buckle and place stretcher in backpack.

### **Guarantee**

This product has 3 year guarantee against all material or manufacturing defects under proper usage and storage conditions. Guarantee is no more valid if; product is used wrong, tears, cut, corrosion, unauthorised repair of changes on the product, damages caused by accidents.

### **Certification**

\* KAYA has Module D Quality System Certificate according to PPE Regulation; 2016/425 EU by APAVE EXPLOITATION FRANCE SAS CE 0082 and confirms each product same as the tested sample.

**Warning:** Read user guide carefully before usage.

\* You can reach the DoC of product on our website [www.kayasafety.com](http://www.kayasafety.com)

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APAVE EXPLOITATION FRANCE SAS - N°0082